

Installation ergoselect II USB driver


USB INTERFACE

Before the USB interface to a PC can be used, a driver has to be installed, which generates a virtual COMx (serial) port in the PC.

DRIVER INSTALLATION (VIRTUAL COMx)

Administrator rights are required to install the driver.

- Start PC and Monitor.
- Finish all running programs (- in background running programs as well).
- Insert the USB driver CD into the CD-ROM drive.
If the installation does not start automatically, start the program „setup.exe“ from the windows explorer.
- Follow the instructions given on the screen.
Confirm the installation if a warning for a non identified program is displayed.
- Click „Finish“ to start the first part of the USB driver installation.
- Connect the ergoselect ergometer with the USB cable to the PC and switch the ergometer on:
the ergoselect ergometer is automatically recognized by WINDOWS and the driver is loaded („Silicon Labs CP 210x“).
- Follow the instructions given on the screen.
- Remove the CD from the drive unit.

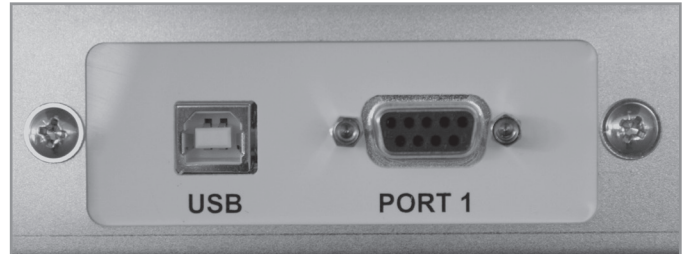

CHECK USB INTERFACE

The ergoselect ergometer has to be connected to the PC via USB cable and switched on.

- Start the WINDOWS „Device manager“.
- Double click „COM & LPT“ to display all connections.
- If an USB Serial Port („Silicon Labs CP 210x USB to UART Bridge“) is assigned no change is necessary.
Keep the assigned COM port in mind - this port number has to be used in the settings of the program controlling the ergometer.
- Close all open windows.